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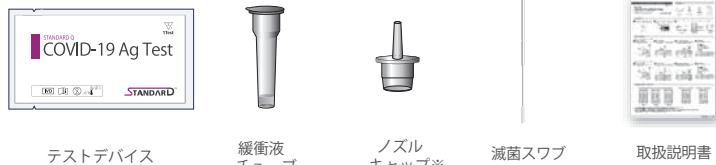
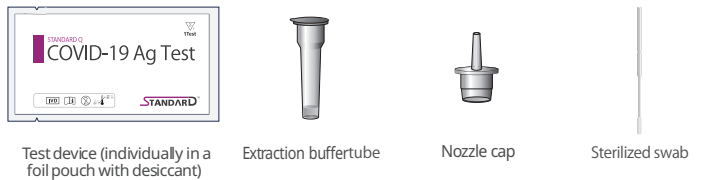
STANDARD™ Q COVID-19 Ag Home Test

PLEASE READ CAREFULLY BEFORE YOU PERFORM THE TEST


IVD Manufacture and Sales Approval Number 30600EZ00016000

KIT CONTENTS

PREPARATION

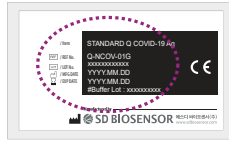
- Carefully read instructions for using the STANDARD Q COVID-19 Ag Test.
- Check the expiry date at the back of the foil pouch. Do not use the kit, if expiry date has passed.
- Check the test device and the desiccant pack in the foil pouch.

<Foil pouch>
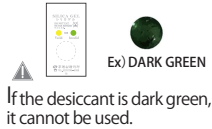
<Test device>


<Desiccant>





How to use

Collect the nasal swab using a sterile swab

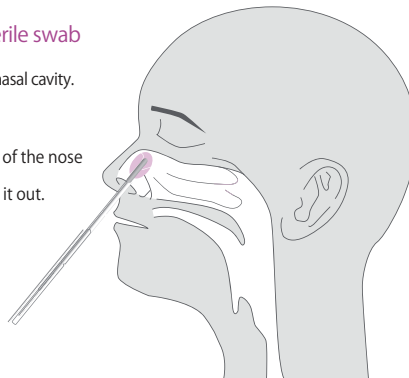- Insert the swab about 2 cm along the nasal cavity.
- Rotate the swab along the inner wall of the nose about 5 times. Leave it for about 5 seconds and pull it out.

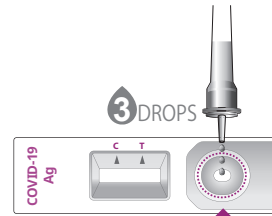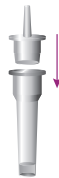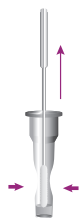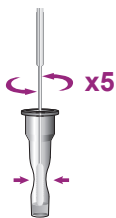- Insert the swab into an extraction buffer tube. While squeezing the buffer tube, stir the swab more than 5 times.
- Remove the swab while squeezing the sides of the tube to extract the liquid from the swab.
- Press the nozzle cap tightly onto the tube.
- Apply 3 drops of extracted specimen to the specimen well of the test device.
- Read the test result in 15-30 minutes.


Read in 15-30 mins.
Do not read after 30 mins.

CAUTION Do not read test results after 30 minutes. It may give false results.

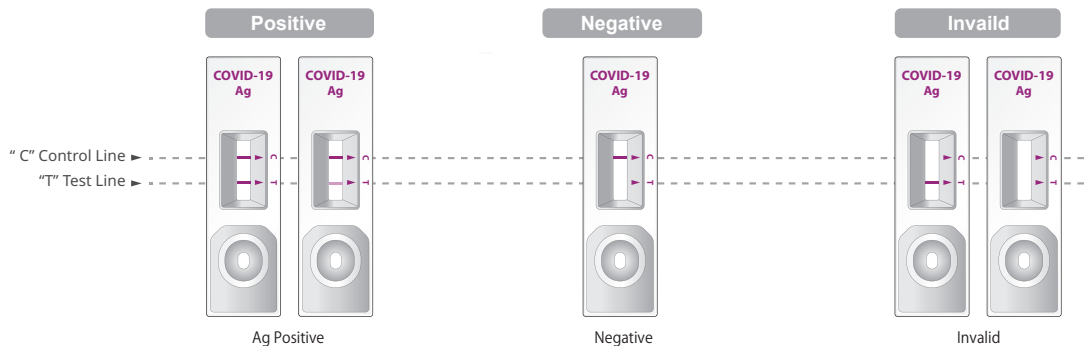INTERPRETATION OF TEST RESULT

- If you get a positive result, you need to see a medical institution.
- Even if the result becomes negative, there is a risk of "false negative" that gives a negative result even though the virus is infected with a small amount, so it is necessary to continue infection control measures such as wearing a mask.

Manufacturer

C-4&5Floor, 16, Deogyong-daero 1556beon-gil,
Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690,
REPUBLIC OF KOREA
Tel +82-31-300-0400 | Fax +82-31-300-0499

Manufacture and Sales

MALCOM CO.,LTD

15-10, HONMACHI 4-CHOME, SHIBUYA-KU, TOKYO,
151-0071, JAPAN
e-mail: mdinfo@malcom.co.jp
URL: http://www.malcom.co.jp